tobii
dynavox

TD Snap Neuerungen 24/25

Sven Pfeil
/ Trainer, Logopäde

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


The screenshot displays two search windows. The top window, titled 'Symbolsuche', shows search results for the word 'sprechen'. It features a search bar with the text 'sprechen', a filter menu with 'Symbol' selected, and a grid of 354 results. Each result consists of a small icon of people talking and the word 'sprechen' written below it. The bottom window, also titled 'Symbolsuche', shows search results for 'Rewe-Flag Herleburg'. It has a search bar with the text 'Rewe-Flag Herleburg', a filter menu with 'Web' selected, and a grid of 100 results. These results include various images of Rewe stores, product packaging like 'Arndorfer' ice cream, and other related content. Three red boxes highlight specific elements: the search bar of the bottom window, the 'Web' filter button, and a small text box at the bottom left that reads 'Ergebnisse aus Microsoft Bing My search: Rewe-Flag Herleburg'.

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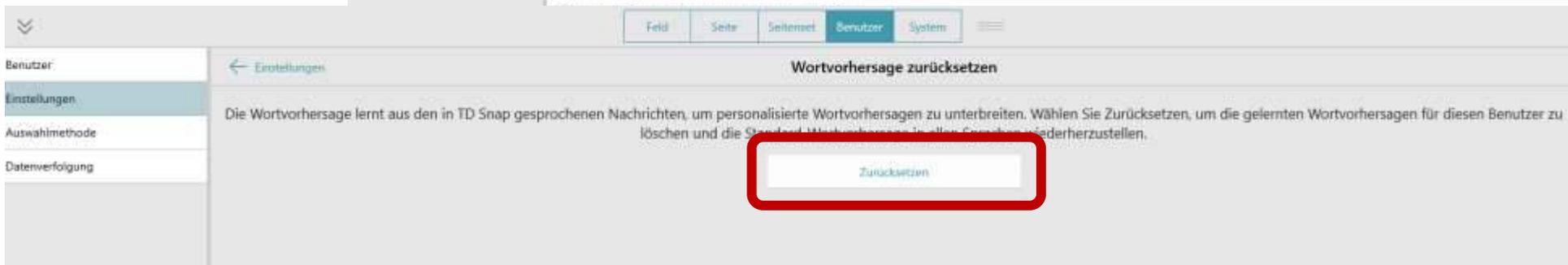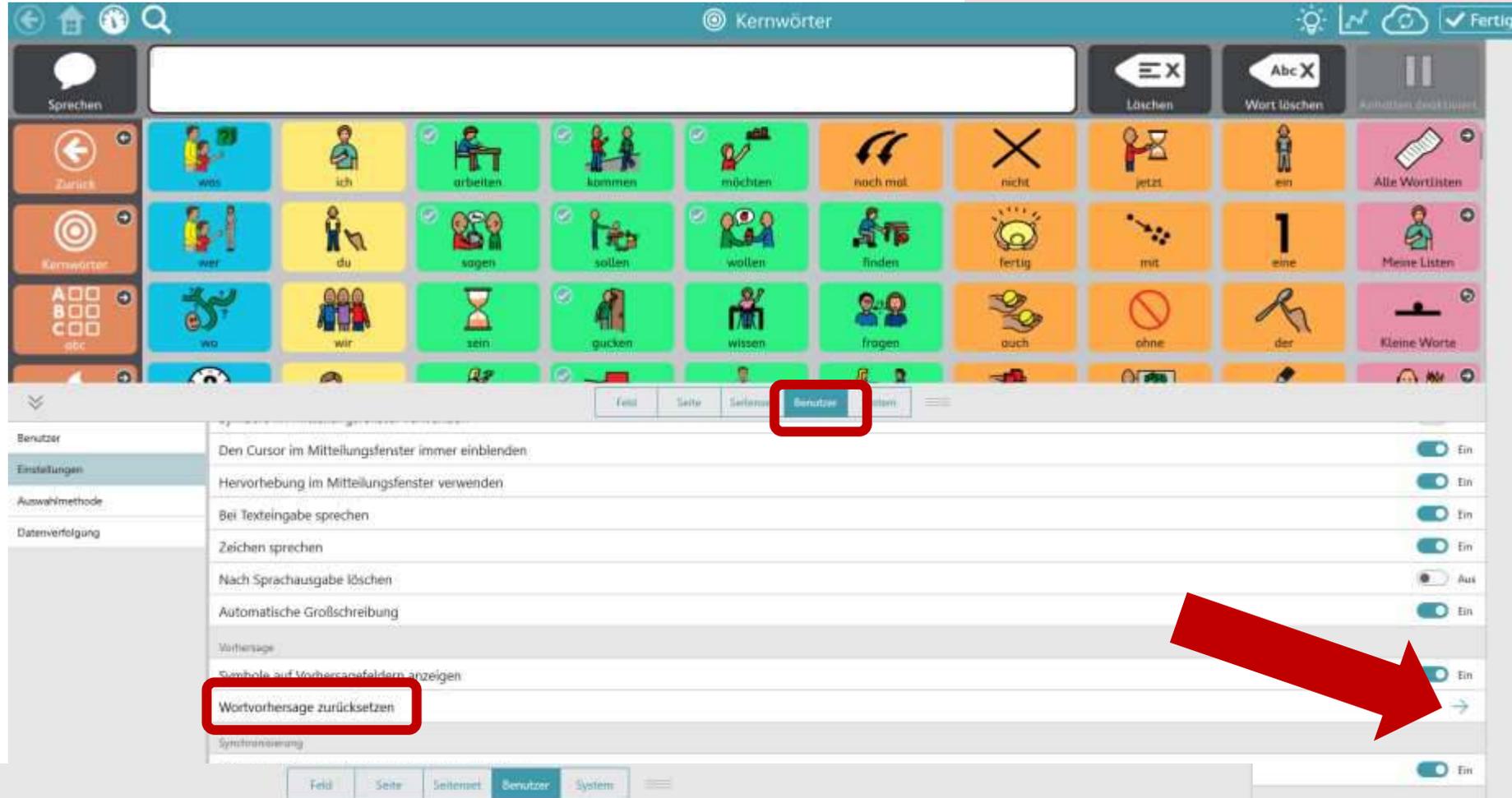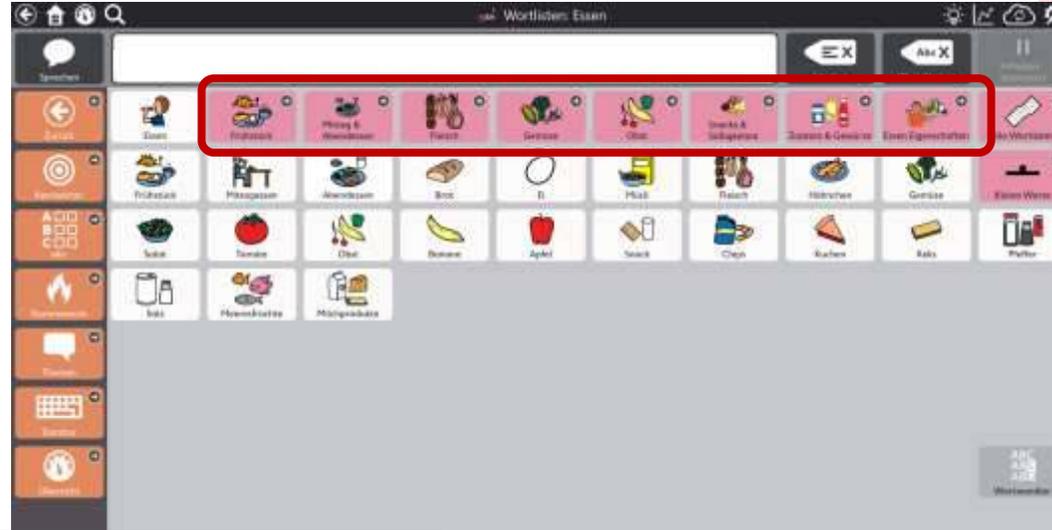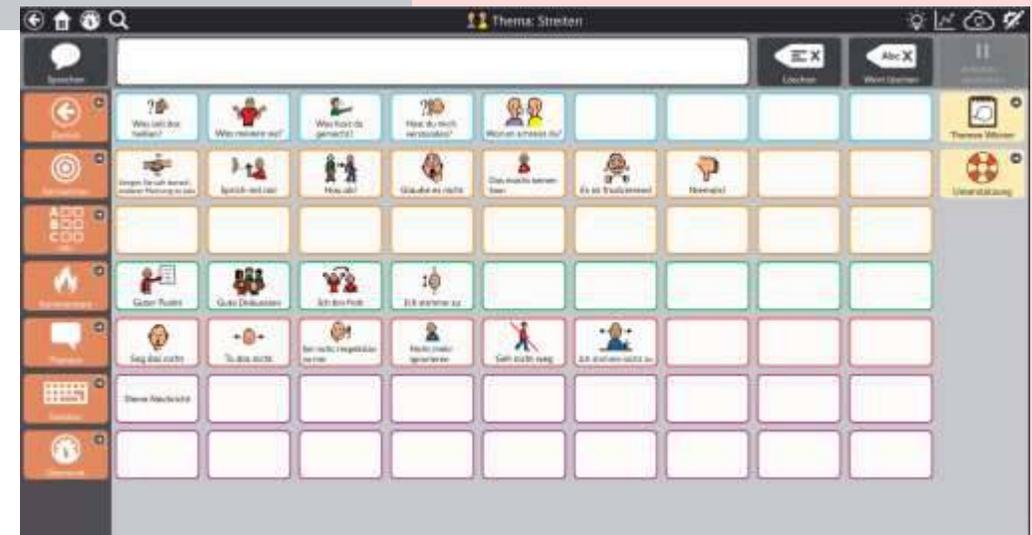
The screenshot displays the 'Hauptseite' (Main page) of the software, featuring several application tiles: 'Core First PCS', 'Core First METACOM', 'Express', 'Voice Smileys', and a grid icon. Below the tiles is a navigation bar with 'Feld', 'Seite', 'Sitzraum', 'Benutzer', and 'System' tabs. The 'Benutzer' tab is active, showing a settings menu with the following items:

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The 'Audiofeedback' settings are expanded, showing the following options:

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